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INSTRUCTIONS FOR USE – ENGLISH
<p>THANK YOU FOR CHOOSING THE RC-CHAMBER® THERAPY.</p> <p>NOTE</p>

Please read the Instructions for Use carefully before using **RC-Chamber®** for the first time. Please also note the instructions for the metered-dose inhaler. Please keep these instructions for future reference. Please note that the device is designed for single-patient use only! All **RC-Chamber®** parts can be sterilized in boiling water and disinfected in a steam sterilizer as well as cleaned in the RC-Clean® cleaning bag. (To prevent the build-up of lime-scale, we recommend boiling the device in distilled water.) Store **RC-Chamber®** in a dry, dust-free environment and protect it from sunlight. We recommend the exchange of the **RC-Chamber®** after one year of using. In Germany the **RC-Chamber®** is reimbursable once a year on submission of a medical prescription.
Disposal: **RC-Chamber®** and its accessories may be discarded with household waste unless otherwise stipulated by the regulations of the respective country of use.

⚠️ WARNINGS

Do not allow children to use with **RC-Chamber®** without supervision. An adult should always be present when therapy is administered to children and those requiring assistance. The tool contains small parts. Please keep it well away from babies and small children in order to avoid the risk of choking. Please clean the device prior to using it for the first time. Clean the device once per week if used regularly, or more frequently as needed. If the device is visibly soiled, clean **RC-Chamber®** immediately prior to or after use. Replace **RC-Chamber®** after one year if used daily. Please ensure that **RC-Chamber®** is used properly according to the Instructions for Use. Incorrect application (off-label use) results in exclusion of liability for the manufacturer.

CONTENTS

The package contains RC-Chamber®, consisting of:

- a. Mouthpiece
- b. Valve & inhalation controls - Cross Valve Technology™
- c. Spray entrance chamber
- d. Back piece: clear plastic ring with integrated rubber adapter
- e. A mask is also included with the following articles:
 - RC-Chamber® with mask for infants
 - RC-Chamber® with mask for young children
 - RC-Chamber® with mask for adults and older children
- f. Emergency plan (to stick to the back of the security bag)
- g. Guardian angel pendant (to attach to the security bag, perforation above the pressure lock necessary)

Materials:
Chamber: Makrolon polycarbonate resin
Back piece: Santoprene
Valve & inhalation controls as well as masks: Silicone
All materials are latex-free.

Safety breathing concept

A reliable companion throughout the day and in an emergency

<p>Included in delivery RC-Chamber® including emergency plan and guardian angel pendant. Drugs must be ordered separately.</p>	
<p>Further information and films on the RC-Chamber® on www.cegla.de</p>	

AVAILABLE PRODUCTS

RC-Chamber® with mask for infants up to 1 year
Article no. 30100 | PPN 111 171 099 278
RC-Chamber® with mask for young children, 1-5 year
Article no. 30200 | PPN 111 171 100 087
RC-Chamber® with mouthpiece for adults and older children
Article no. 30300 | PPN 111 171 102 343
RC-Chamber® with mask for adults and older children
Article no. 30400 | PPN 111 171 101 777
RC-Chamber® for tracheostomized patients
Article no. 30450 | PPN 111 273 164 894

MASKS

Maskes available separately.
Infant mask
Article no. 30110 | PPN 111 416 498 106
Child mask
Article no. 30210 | PPN 111 416 499 893
Mask for adults and older children
Article no. 30410 | PPN 111 416 500 605
RC-Tracheal mask for tracheotomized patients
Article no. 3012

ACCESSORIES AND ADDITIONAL PRODUCTS

RC-Clean®
Reusable microwaveable cleaning bag. Easy and quick cleaning of medical products as well as baby items.* Use only for microwave safe products.
Article no. 5000 | PPN 111 075 103 265
* Report on microbiological testing available on request.

Replacement parts and additional products are available in pharmacies and medical supply stores.

To constantly improve our level of service, we welcome your feedback. You will find helpful tips for living with respiratory diseases on our platform www.leichter-atmen.de. We look forward to your visit.

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PURPOSE

RC-Chamber® is an inhalation device developed specifically for medication spray therapy, so-called MDIs, metered-dose inhalers, and is used in the treatment of respiratory diseases. The flexible rubber adapter is compatible with all conventional dose inhalers.

When using **RC-Chamber®**

- it is no longer necessary to coordinate the release of the aerosol spray and inhalation of the drug
- large, non-respirable particles are filtered out. This helps prevent irritation that may lead to yeast infections in the mouth and throat.

A Without **RC-Chamber®** most of the drug is deposited in the mouth and throat.

B With **RC-Chamber®** delivery of the drug to the lungs improves.

INSTRUCTIONS Use

Please inspect the device for foreign bodies and damage prior to use. If any parts are missing or damaged, please replace them immediately.

Attach the mask securely until it locks in.

Remove the sealing cap from the MDI mouthpiece and shake the aerosol spray container vigorously prior to each use.

Place the MDI mouthpiece as shown in the illustration into the rubber adapter provided on **RC-Chamber®**.

Make sure the connection to the rubber adapter is straight and tight.

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Tip

To familiarize children with the use of **RC-Chamber®**, it may help to release the aerosol spray into the chamber before putting the mask on the child.

RC-Chamber® for tracheostomized patients

Connect **RC-Chamber®** directly to the cannula or to the **RC-Mask Tracheostoma** in the event of an open tracheostoma.

RC-Chamber® with mouthpiece

- I. Push the cartridge of the MDI down in order to release the spray into **RC-Chamber®**.
- II. Inhale and hold your breath for 5 to 10 seconds. This optimizes the therapeutic effect of the spray.

RC-Chamber® with mask

- I. Push the cartridge of the MDI down in order to release the spray into **RC-Chamber®**.
- II. Hold the **RC-Chamber®** mask combination to the face for 4 to 6 breaths. This procedure can even be administered to a sleeping child. The number of breaths can be seen in the valve movement.

RC-Chamber® for tracheostomized patients

- I. Push the cartridge of the MDI down in order to release the spray into **RC-Chamber®**.
- II. Inhale and hold your breath for 5 to 10 seconds. This optimizes the therapeutic effect of the spray.

If the treating physician prescribed several spray applications, always only release one puff into the chamber for every inhalation procedure. Please wait approximately one minute before repeating steps IV to VI!

INSTRUCTIONS Cleaning

After using a mask, detach it from **RC-Chamber®**. Remove the clear plastic ring with the attached integrated rubber adapter by pressing your thumb deep into the opening. Use force to pull off the back piece.

Cleaning and disinfection

1. **Wash** the parts in a solution of lukewarm clean water and standard liquid soap detergent. The **RC-Chamber®** is dishwasher safe. Due to the possibility of leftovers we do not recommend this cleaning method.
2. **To disinfect** all parts, place them in boiling water for approx. 5 minutes. Please use a clean pot with sufficient fresh drinking water.

The **RC-Clean®** cleaning bag may also be used to clean the device (see accessories).

Shake to remove excess water and allow all parts to air-dry completely before reassembly. **Do not wipe dry!**

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Reassemble when all parts are completely dry. Attach the mask securely until it locks in. Make sure that the rubber adapter is attached in the same direction as the mouthpiece. Store **RC-Chamber®** with or without mask in a dry, dust-free environment, for example in the original **RC-Chamber®** package. Protect from sunlight.

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