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⚠️ WARNING | INSTRUCTION FOR USE

- Fill the cleaning bag RC-Clean only with microwave durable objects. Do not put any metal pieces, pointy objects or electronic items in the bag.
- Do not use the microwave's grill function or a conventional oven.
- Always fill the RC-Clean with 60 ml of water. We recommend the use of distilled water.
- In addition, please observe the instructions for use of your microwave.
- The cleaning bag can be reused up to 20 times. Check off after each use.
- Keep the Cleaning Bag out of the reach of children – risk of suffocation!

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Don't clean any metal parts, pointed or electrical objects in the bag!

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Cleaning Bag for the Microwave

Clean medical products and childcare items easily and quickly.

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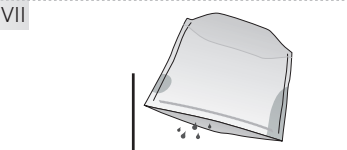
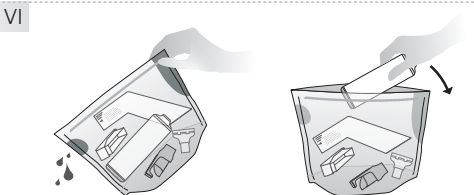
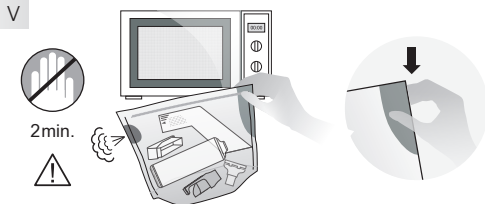
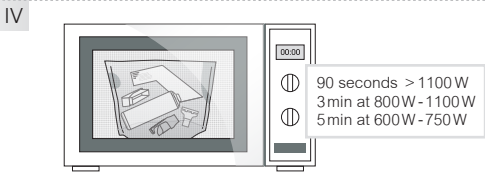
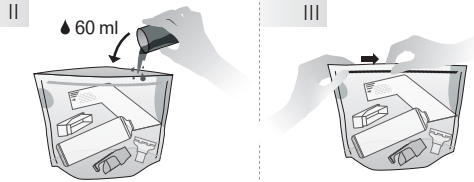
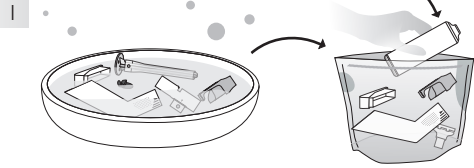
INSTRUCTION FOR USE

I Clean all parts with warm soapy water and rinse them with clear water before use. Place the products which you wish to clean in the RC-Clean bag. Do not overfill the bag. **Do not put any metal pieces or electronic items in the bag.**

II Always pre-fill the RC-Clean with 60 ml of water. We recommend the use of distilled water.

III Seal the cleaning bag with the pressure lock. The bag must be firmly sealed.

IV Lay the cleaning bag in the middle of the microwave. Adjust your microwave to the correct wattage and duration and start the microwave according to the following table:
 90 seconds at > 1100 W
 3 minutes at 800 W - 1100 W
 5 minutes at 600 W - 750 W

V Open the microwave after the time has expired and let the RC-Clean cool for a minimum of 2 minutes. Take hold of the bag only by the safety zone and take it out of the microwave. Please note that when taking out or opening the RC-Clean hot steam or water can escape.

VI Pour the excess water carefully out of the cleaning bag through the steam valve holding by the Safety Zone. Open the reusable bag to remove the cleaned pieces. Ensure you wash your hands prior to handling the cleaned pieces. Allow all the pieces air dry completely. Do not rinse or rub dry. The cleaned pieces are now ready for use.

VII After using the cleaning bag, please turn upside down and allow it to dry out.

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